CATALOGO
CATALOGUE

FRESE DIAMANTATE STERILI MONOUSO

DISPOSABLE STERILE DENTAL DIAMOND BURS

CE0051

**DIAMIR**

Frese diamantate sterili monouso

Disposable sterile dental diamond burs


Forma/Shape	Codice DIAMIR DIAMIR Code	Quantità Quantity
	M3D 801010	
	M3D 801016	
	M3D 805012	
	M3D 805016	
	M3D 806009	
	M3D 830012	
	M3D 835008	
	M3D 835014	
	M3D 837012	
	M3D 838010	
	M3D 845014	

Forma/Shape	Codice DIAMIR DIAMIR Code	Quantità Quantity
	M3D 847014	
	M3D 850016	
	M3D 852012	
	M3D 856012	
	M3D 858014	
	M3D 859012	
	M3D 860012	
	M3D 862014	
	M3D 881014	
	M3D 368010	
	M3D 368018	

Massima sicurezza • High safety

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Complete sterilization of the bur is guaranteed according to official Pharmacopeica standards. The special method of packaging permits the bur to be inserted in the drill without the operator having to touch it.

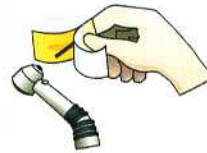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In packages of 60 pieces of the same shape

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Indications and instructions for use

To get the best from **DIAMIR** instruments, the instructions listed herewith should be followed carefully:

- Use suitable cooling methods to guarantee correct cooling of the parts subject to treatment.
- Use aspiration techniques which do not compromise cooling of the parts involved in the treatment.
- Use only handpieces in perfect condition, as well as cooling nozzles in perfect operation.
- Check that the package has not been tampered and the expiration date.
- Use only tools which are in perfect condition, not only in terms of the diamond edge but also as regards concentricity.
- Insert the instruments very carefully and without forcing.
- Before using, check that the tool shank locks perfectly with the handpiece.
- Bring the tool to the necessary speed before approaching the part to be treated.
- Use a pressure force between 0.3 - 2 N (approximately 30 - 200 g). Under no circumstance must greater pressures be applied.
- Avoid using the tools as levers and/or at incorrect angulations.
- Respect the speeds shown in the attached table.
- Throw the instrument after use.

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