STERILE
SINGLE USE ONLY

3D

STERILE
SINGLE USE ONLY

3D

60 PCS
REF M3D8350

STERILE R LOT 1210

2012/04

CE 0051

DIAMIR srl

Via Prato 2

I - 33010 R

ITALY

www.diamir.it

Frese Diamantate Sterili Monouso
Disposable Sterile Dental Diamond Burs

DIAMIR

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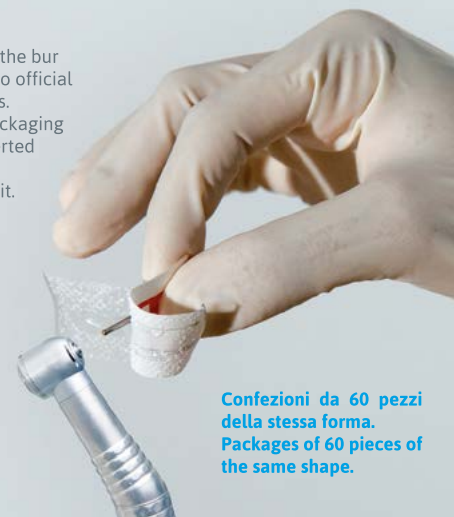
Complete sterilization of the bur is guaranteed according to official Pharmacopoeia standards. The special method of packaging permits the bur to be inserted in the drill without the operator having to touch it.

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DIAMIR dental diamond burs - Sterile - Single Use Only 

Field of use

Preparation of the cavity (preservative)
Preparation of the stumps (prosthetic)
Finishing of the cavity edges and repair work
Polishing of the repairs

Instructions for use

To get the best from **DIAMIR** instruments, the instructions listed herewith should be followed carefully:

- Use suitable cooling methods to guarantee correct cooling of the parts subject to treatment.
- Use aspiration techniques which do not compromise cooling of the parts involved in the treatment.
- Use only hand pieces in perfect condition, as well as cooling nozzles in perfect operation.
- Use only tools which are in perfect condition, not only in terms of the diamond edge but also as regards concentricity.
- Insert the instruments very carefully and without forcing.
- Before using, check that the tool shank locks perfectly with the hand piece.
- Bring the tool to the necessary speed before approaching the part to be treated.
- Use a pressure force between 0.3 - 2 N (approximately 30 - 200 g). Under no circumstance must greater pressures be applied.
- Avoid using the tools as levers and/or at incorrect angulations.
- Respect the speeds shown in the attached table.
- Check that the package has not been damaged and the expiration date.
- Single use only. Do not reuse.

Advice and Recommendations on Hygiene

It is pointed out that the products do not have to be reused.

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