STERILE
SINGLE USE ONLY

D

STERILE
SINGLE USE ONLY

D

60 PCS
REF M3D8350

STERILE R LOT 120

2012/04

CE 0051

DIAMIR srl

Via Prato 2

I - 33010 R

ITALY

www.diamir.it

Frese Diamantate Sterili Monouso
Disposable Sterile Dental Diamond Burs

DIAMIR

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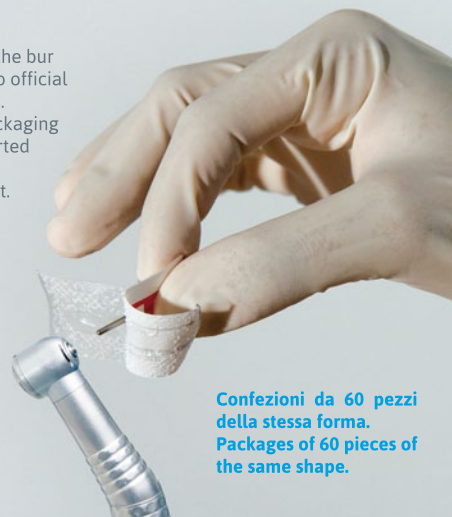
Complete sterilization of the bur is guaranteed according to official Pharmacopoeia standards. The special method of packaging permits the bur to be inserted in the drill without the operator having to touch it.

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Indications and instructions for use

To get the best from **DIAMIR** instruments, the instructions listed herewith should be followed carefully:

- Use suitable cooling methods to guarantee correct cooling of the parts subject to treatment.
- Use aspiration techniques which do not compromise cooling of the parts involved in the treatment.
- Use only handpieces in perfect condition, as well as cooling nozzles in perfect operation.
- Check that the package has not been tampered and the expiration date.
- Use only tools which are in perfect condition, not only in terms of the diamond edge but also as regards concentricity.
- Insert the instruments very carefully and without forcing.
- Before using, check that the tool shank locks perfectly with the handpiece.
- Bring the tool to the necessary speed before approaching the part to be treated.
- Use a pressure force between 0.3 – 2 N (approximately 30 – 200 g). Under no circumstance must greater pressures be applied.
- Avoid using the tools as levers and/or at incorrect angulations.
- Respect the speeds shown in the attached table.
- Throw the instrument after use.

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